

Standard Warranty: Dencept Industries, LLC warrants that this product shall be free from defects in materials and workmanship for one year. If the product is defective, Dencept Industries, LLC's sole obligation under this warranty shall be at its election, subject to the terms of this warranty, to repair or replace any parts deemed defective. This warranty is for the exclusive benefit of the original purchaser and is not assignable or transferable.


THIS WARRANTY IS MADE IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED AND ANY IMPLIED WARRANTIES, INCLUDING THE IMPLIED WARRANTIES, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE TERMS OF THIS WARRANTY, UNLESS OTHERWISE PROHIBITED BY LAW.

Exclusions: To the fullest extent permitted by law, Dencept Industries, LLC expressly disclaims and excludes from this warranty any liability for damages, including but not limited to general, special, indirect, incidental consequential, aggravated, punitive or exemplary damages, and economic loss (even if Dencept Industries, LLC is informed of their possibility), third party claims against you, include for any loss or damages, and claims or damages related to loss of, or, damage to, your records or data. This warranty will be voided by misuse, improper physical environment, improper repair, act of God, war or terror, and improper service, operation or handling.

Warranty Return Merchandise: Any claim under this warranty must be made in writing to Dencept Industries, LLC or by calling technical support, within 30 days after the owner discovers the circumstances giving rise to any such claim, and must include a copy of warranty document, a receipt or copy of your invoice marked with the date of purchase, and the original packaging. You may obtain service by calling the technician. Package must be mailed to Dencept Dental, postage prepaid.

Return Policy: Returned Merchandise other than for warranty is subject to a 25% restocking fee within 30 days of purchase. After 30 days returns will not be accepted.

Find instructional
videos for
different software
tips and tricks, at
DenceptDental.com

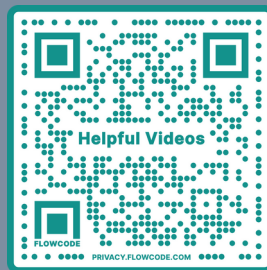

DENTAL R.A.T.

THE HANDS-FREE CHARTING SYSTEM



877-278-0061
support@denceptdental.com
DenceptDental.com



DENTAL R.A.T. 2.0 WIRELESS INSTALLATION

- 1) Plug the USB dongle into the computer's USB port. The Dental R.A.T. 2.0 will automatically upload itself.
- 2) Put (3) AA Batteries into the back of the unit. With the heel at the bottom, the **batteries should be - + - at the top.**
- 3) Your Dental R.A.T. is ready to operate! Watch the training videos at denceptdental.com to learn tips and tricks for use with your software.

If your R.A.T. isn't working, put the dongle in the USB port, then hold the little button on the dongle. It will blink RED, then flash GREEN, release button. Once that happens, hold the R.A.T. about 6" away and begin to move buttons on the R.A.T. until the cursor on the screen moves.

AUDIO FEEDBACK OPTION

Audio Feedback is included in your system, please visit DenceptDental.com to download the software.



LEFT CLICK

The bottom button acts as a heel rest but is also the left mouse click. This allows the cursor to click on items just as if you were to left mouse click with a hand operated mouse. To activate the left mouse click "rocking back" by lifting the front of your foot. Take care NOT to press down directly as this could damage the unit.



ADJUSTING MOUSE CURSOR SPEED

To speed up - Press & Hold numbers 1 & 2 simultaneously on the unit, while holding the numbers, press the mouse cursor up until you reach the desired speed.

To slow down - Press & Hold numbers 1 & 2 simultaneously on the unit, while holding the numbers, press the mouse cursor down until you reach the desired speed.

NUMBERS

The four buttons across the center are numbers 1-8. If each button is pushed down for less than one second they are 1 - 4. If each button is pushed down for longer than one second they become 5 - 8. (For 9 and up, please watch the training video found on denceptdental.com).

MOUSE

Joystick at the top, moves your mouse cursor.

SERIAL

MONTH/ YEAR OF PURCHASE

WARRANTY EXPIRATION

WARRANTY EXTENTION PURCHASED? YES NO
